

Praxis fuer logopaedie, Wibke kukler

Practice for logopedics  
Wibke Kukler

## Functional range of the Novafon in logopedics

For several years I have been using the Novafon the most diverse areas of logopedic therapies.

### Dysphagia

The Novafon can be used externally as a preparatory stimulating measure in increasing awareness in muscle groups of the oral-facial area in the treatment of dysphagia (increasing sensory perception, eutony).

On the floor of the mouth  
masseter m  
orbicular oris  
levator labil, risorius etc.,

Often, improved oral-facial activity can be observed after or during use, as well as a significantly increased frequency in swallowing following stimulation of the floor of the mouth.

I employ the Novafon when introducing active dysphagia therapy and in the treatment of the oral-facial tract.

### Apallisches Syndrom

Important: Consultation with the physician required concerning the risk of seizure!  
In principle, use is the same as with dysphagia with the difference that prior to using the Novafon at all in the field of basal stimulation according to Prof. Frohlich, I approach the oral-facial area with caution.

Then usage in the same manner as with dysphagia.

Apart from that, on the back to promote expectoration and deep respiration.

### Usage in voice therapy

Apply to acupressure points in order to accelerate loosening and opening the jaw.

To relax and loosen the neck and shoulder area.

On the diaphragm, guiding sensual perception, deep breathing.

On the sternum.

Directly on the larynx, in eutony, and to loosen the voice.

It would be easy to increase the list of the Novafon's applications in logopedics.

Wibke Kukler